BIOTECH DENTAL CONDOR

BROCHURE

BIOTECH DENTAL

BIOTECH DENTAL

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FAQ

➤ WHY ARE YOU LAUNCHING A NEW INTRA-ORAL SCANNER?

The Condor® Intra-oral scanner is a real diagnostic tool for the clinician. Information sharing is easily possible with patients and peers. Furthermore, the scanner facilitates dental impressions and saves valuable time on each step.

➤ WHAT ARE THE ADVANTAGES OF THE CONDOR® INTRA-ORAL SCANNER?

The Condor® has the following distinct benefits:

- **Unique colored 3D image:**

The realism and detailed colored images enable you to have more than just an impression taking device. Easily distinguish tooth structure, restoration, tissue, or blood for locating exactly the margin line or undercuts. It may allow making the first diagnosis without seeing the patient directly or discussing a therapy with peers online while sharing the digital intra-oral situation. Easing communications to educate the patient about treatment and improving patient communication.

- **Powder free scanning:**

Professionals no longer have to use powder spray to prepare teeth before scanning which is uncomfortable for the patient and time consuming. Furthermore powder can undermine scan accuracy.

- **Ergonomic:**

Small and lightweight, the scanner adapts perfectly to the dental unit with your other instruments. The handpiece's thin scanning head does not bother the patient and adapts to a smaller mouth opening.

- **Easy to use:**

System ready for use, very compact scanner allows a «pen» slot to be used. Very easy to handle.

- **Ergonomic user interface:**

The procedure, flexible and intuitive, is simplified.

- **Open System:**

The Condor® produces an open .stl or .ply file, which is compatible with every open computer Aided Design CAD or planning application. Easily archive, send and share the data file with labs and peers.

- **Affordable price**

➤ **WHAT TYPE OF CONFIGURATION WILL THE CONDOR® INTRA-ORAL SCANNER SUPPORT?**

The Condor® Scanner comes with a laptop. It is plugged directly into the computer using a USB 3.0 port.

For better convenience, the handpiece fits perfectly into the dental unit.

➤ **HOW MANY 3D IMAGES CAN BE STORED WHEN USING THE CONDOR® INTRA-ORAL SCANNER?**

The number of 3D images that can be stored are based on the PC's hard disc capacity. But it is easy to export the files to an external hard disk or server.

➤ **WHAT TYPE OF IMAGES CAN BE STORED?**

The Condor® operating system generates as output an open .stl or .ply file.

➤ **HOW CAN BE THE DATA SHARED WITH DENTAL LABS OR PEERS?**

The .stl or .ply output is easily sent by e-mail, uploaded on a portal or ftp-server, or any other method used by your practice due to it being built on the Linux Operating System.

➤ **CAN A DENTAL LAB USE THE DATA IN THEIR CAD APPLICATION?**

If the dental CAD Software used by the lab allows importing an open .stl file, the Condor output can be used easily to design various restorations or appliances (braces, veneers, surgical guides, crowns and bridges, etc.).

➤ **WHAT IS AN OPEN SYSTEM OR OPEN .STL FILE?**

An open system allows importing data from compatible third party software and also generating an open output file. The most common 3D file format for dental CAD CAM is an .stl file. Open .stl file means, that there is no additional encodings and the compatibility is warranted.

➤ **IS THE CONDOR® SCANNER ABLE TO CONNECT WITH A MILLING MACHINE?**

In principle there is no limitation. The Condor® data output can be used in an open CAD CAM application. The designed file can be sent then to a milling machine or 3D printer.

➤ **WITH WHICH OPERATING SYSTEM, THE CONDOR® COMPATIBLE?**

To ensure quality and optimal scanning speed, the Condor® intra-oral scanner unit is provided with a powerful Linux laptop.

➤ **HOW DOES THE CONDOR® INTRA-ORAL SCANNER WORK WITH A PC?**

For high speed data transfer, the Condor® intra-oral scanner connects directly to the PC with USB 3.0 port without the need for a docking station or an additional power cable.

➤ **WHICH TECHNOLOGY OPERATES THE CONDOR® INTRA-ORAL SCANNER?**

The Condor® intra-oral scanner is based on a video photogrammetry system and software exclusive exploitation.

➤ **WILL THE CONDOR® INTRA-ORAL SCANNER COME WITH A FOOT PEDAL?**

There is no need for a foot pedal thanks to a button system on the hand piece and the ergonomical Condor® operating software.

➤ HOW CAN THE CONDOR® INTRA-ORAL SCANNER BE CLEANED OR DISINFECTED?

A disinfectant solution can be used to clean the scanner.
The chosen material is resistant to bacteria for the handpiece.

➤ IS THE CONDOR® INTRA-ORAL SCANNER DELIVERED ALONE?

The Condor® Scanner comes with Linux PC, integrated user interface, and a handpiece.

➤ WHAT WILL BE THE CONDOR® WARRANTY?

A 2 years warranty is standard.

➤ WHO WILL INSTALL AND SUPPORT THE NEW SCANNER ?

Due to the extreme simplicity of the Condor® Scanner, the installation and usage is very simple. In case of questions, tutorial videos and easy-to-understand quick guides are provided. In addition, the certified distributor will provide training, full service and support to guarantee the Condor® intra-oral scanner experience.

➤ HOW MUCH IS THE ANNUAL LICENSE OF THE CONDOR® SCANNER?

The Condor® intra-oral scanner is associated with no annual fee or compulsory license.

Biotech Dental Vietnam

407/8 Sư Vạn Hạnh, P. 12, Q.10 - Tp.HCM

+84 (0) 28 38 620 192

+84 (0) 917 96 09 17

customers.care@nidp-botechdental.com.vn

www.nidp-botechdental.com.vn

UPPERSiDE

BIOTECH DENTAL

BIOTECH DENTAL
VIETNAM

Manufactured by: CONDOR - Distributed by: Biotech Dental